1. **Q:** Which finger should be used to obtain best results?  
**A:** Use any finger that easily fits into the thimble; do not force the finger into the thimble.
2. **Q:** After I start the Unit, only dashes appear.  
**A:** This is normal for the first 20 seconds. Ensure that the display is on the nail side of the finger, as indicated in the user instructions. If the reading takes longer than 20 seconds to appear, it may be indicating that the Unit is too tight on the finger, causing impairment or restriction of the circulation. Pull the finger out a little and the Unit will restart. If the second try is unsuccessful, repeat the insert/start process on a smaller finger. If the user has very small fingers (e.g. children), try using the thumb.
3. **Q:** I get a “weak signal” icon, what does this mean?  
**A:** After starting and getting a good reading, if the sensor does not receive a signal for more than about 5 seconds, this icon will appear, even though the LCD continues to show its last update. It may be indicating that the unit is too tight on the finger and restricting or stopping circulation – refer to FAQ # 2.
4. **Q:** I get a sensor “OFF” icon, what does this mean?  
**A:** Ensure that the display is on the nail side of the finger. The sensor “OFF” may also indicate that the Unit is too tight on the finger and be restricting or stopping circulation - refer to FAQ # 2.
5. **Q:** The illuminating backlight turns off.  
**A:** The light is designed to operate at 10 seconds intervals to conserve on battery life. Press the button for about one second again to turn the light on for another ten seconds.
6. **Q:** How do I change the battery?  
**A:** Using a flat screwdriver, snap open the battery cover on the front of the Unit. Remove the old battery and discard in accordance with local environmental laws. Install a new battery by matching the +/- markings inside the battery compartment and replace battery door. CAUTION: installing an incorrect battery may damage the Unit and void the warranty.
7. **Q:** Where can I purchase a new battery?  
**A:** 3.6V ½ AA batteries are available in many electronics stores, e.g. Radio Shack / USA.  
**A:** Replacement batteries can be purchased from Customer Service (1-805-823-8460) or via email [support@spomedical.com](mailto:support@spomedical.com).
8. **Q:** What is the normal range for each display (SpO<sub>2</sub> and pulse)?  
**A:** Individual normal ranges should be defined by your physician.
9. **Q:** Can I put it on a wet (or bloody) finger?  
**A:** No. This model is not water resistant, and exposure to excessive moisture will damage the Unit and void the warranty. Wipe all moisture from the finger before inserting into the flexible thimble. If finger is injured or bleeding, use a different finger.
10. **Q:** The Unit dropped into the water.  
**A:** Turn off Unit immediately and remove the battery. Most likely, the Unit is damaged and beyond repair. However, you may try to dry out the Unit by shaking out any water and leaving the Unit open without the battery for several days. If your Unit does not function after several days of drying time, discard the Unit at your local electronics recycling center.

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